

Steel tube manual wheelchair [Portable]

鋼管手動輪椅車【輕便款】

Model: LR-WBQ

Features:

- It can bear up to 200 kg. There are 4 brakes in front of and behind it, which is convenient for nursing and safe
- 5 seconds quick folding, just for more convenient travel
- Lengthen the handrail to make it easier to grasp
- Oxford cloth is easy to clean and take care of
- The steel pipe frame is firm and durable, and the double support frame has strong stability
- The foot can be turned up, which is more convenient for getting on and off
- Suitable for people with muscle weakness and mobility difficulties
- 承重可達200斤，前後共4剎車，方便護理，安全放心
- 5秒快速折疊，只為出行更便捷
- 加長扶手，抓握更方便
- 牛津布料方便清洗，易打理
- 鋼管車架堅固耐用，雙支持車架穩固性強
- 腳踏可上翻，上下車更方便
- 適合人群：肌無力，行動不便的人



Specifications

Product Model 產品型號	LR-WBQ
Material 材質	steel pipe 鋼管
Product Function 產品功能	out walking aid, foldable 外出助行,可折疊
Control Mode 控制方式	manual control 手動控制
Front tyre 前輪尺寸	7 inches
Rear tyre 後輪尺寸	22 inches
Product Size 產品尺寸	100*66*88cm
Product Folding Size 折疊尺寸	27*96*88cm
Product Weight 產品重量	14.4kg

Instructions

◆輪椅展開和折疊方法:

1. 雙手握住把手，將車後輪提起離開地面，向兩側輕拉
2. 將左右車架分開（靠墊上沿繃直），向下輕壓坐墊兩側至輪椅展開
3. 將腳踏板放平，系好腿帶
4. 用手提拉坐墊中間，使輪椅車兩側大架慢慢向中間收攏
5. 按住左右扶手外側向內壓合即可

◆注意事項:

1. 向下壓坐墊時切勿將手指放入夾縫中，以免受傷
2. 切勿在未剎住駐立剎車時乘坐輪椅，以免輪椅滑動造成身體傷害
3. 切勿腳踩腳踏板上下輪椅，謹防輪椅前傾造成身體傷害
4. 切勿抓握其他部位抬動輪椅，以免造成輪椅損壞或身體傷害
5. 下坡時，不能使用駐立剎車，以免翻車造成人身傷害
6. 下車時請務必先剎住駐立剎車，駐立剎車是為使用者上下輪椅或停靠時使用，不能用來在行駛中作制動使用